

- ⚠ **WARNING: Important! Read and understand these instructions before installing or using the raised toilet seat. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the raised toilet seat is not properly assembled and installed, personal injury and damage to the raised toilet seat could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.**
- WARNING: This product has a maximum weight capacity of 300 lb, EVENLY DISTRIBUTED.**
- ⚠ **WARNING: ALCO Sales & Service Co. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INTENDED USE


The raised toilet seat is intended for use by those who have difficulty sitting down on or standing up from the toilet.

INSTALLATION

Remove raised toilet seat, hardware, and protective packaging from carton. Remove the hardware bag from inside the seat and shield assembly.


Splashguard models only: If the splashguard is not attached to the raised toilet seat, perform the following steps to attach it (*tools required: Phillips or blade screwdriver*).

1. Remove hardware containing three screws from bag.
2. Align the three splashguard holes with the three holes on the bottom of the toilet seat, as shown at right.
3. Use a Phillips or blade screwdriver to screw the splashguard to the toilet seat with the provided screws. Start on one side and work your way around to align the holes and install the screws.

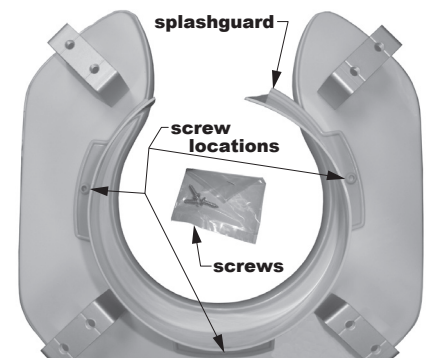

Underside with splashguard

All models: See picture at right of each model with components labeled. Each toilet seat has four rim clamps (two front and two rear) that attach the toilet seat to the toilet rim, and four seat brackets (two front and two rear), into which the clamps screw, that adjust seat height.

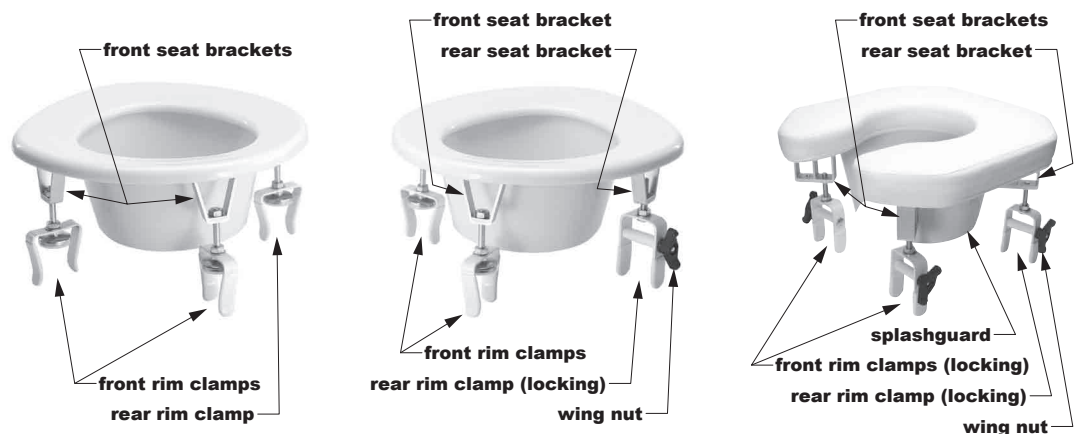

1. Align front rim clamps to toilet seat rim and snap them on. If clamps are too loose, squeeze them closed slightly to ensure a tight fit.
2. Screw the two front rim clamps into the front seat brackets and adjust seat front to desired height. **For stability, ensure that seat front is adjusted to the same height on both sides.**
3. Align back rim clamps to toilet seat rim and snap them on.

4. Screw the two back rim clamps into the back seat brackets and adjust seat back to desired height. ***For stability, ensure that seat back is adjusted to the same height on both sides.***

Info: For certain situations such as postoperative, hip conditions or arthritis, seat back can be adjusted slightly higher than seat front.

Locking Clamp models only: Tighten the wing nuts until the sides of the clamps are secure against the sides of the toilet bowl rim.

OPERATION

⚠ WARNING: Always ensure the raised toilet seat is securely installed and fits tightly on toilet bowl before use.

⚠ WARNING: Users with limited physical capabilities should be supervised or assisted when using raised toilet seat.

MAINTENANCE AND CLEANING

Loosen, retighten, and check for secure fit of raised toilet seat weekly. To clean toilet seat, wash with soap and water or a mild household detergent and rinse with clear water. Ensure the toilet seat is rinsed well before use.

If moisture accumulates inside raised toilet seat, do the following:

1. Remove toilet seat from toilet bowl.
2. Turn toilet seat upside down and tilt toward drain opening.
3. Rotate until drain opening is facing downward and moisture drains from toilet seat.
4. Allow toilet seat to air dry for several hours.

⚠ WARNING: Inspect the toilet seat weekly as described above and tighten it as needed to ensure secure installation.

info@alcosales.com

800.323.4282

www.alcosales.com

