


Assembly instructions for IV Poles

Tools you will need for assembly:

- 9/16" Flat Wrench
1. You should receive in your shipment:
 - a. Base
 - b. Pole
 - c. Threadlocker
 - d. Top

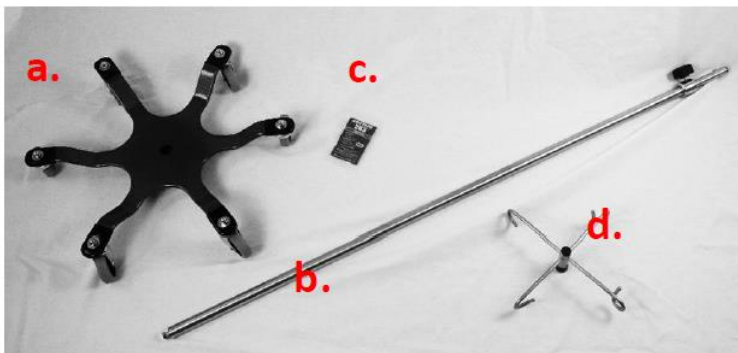

2. Remove all packing material and unscrew Bolt and Washer from the bottom of (b. Pole).
3. Insert bottom of Pole into hole on top of (a. Base)
4. Open (c. Threadlocker) and apply generous amount to threads on Bolt that was removed from bottom of pole.
5. With tooth washer on bolt insert bolt in small hole in middle of (a. Base) from the bottom. Tighten Bolt with 9/16" Wrench. Ensure pole is firmly seated and bolt is tightened as much as possible with wrench.
6. To add top, slide top over top of inner pole. Squeeze buttons with fingers and slide top down. Line up holes so buttons pop through holes in (d. top).
7. Let pole stand for 4-6 Hours for threadlocker to set.
8. Your new IV Pole is now ready for use.