


# AL-50363 OVER BED TABLE ASSEMBLY AND OPERATING INSTRUCTIONS

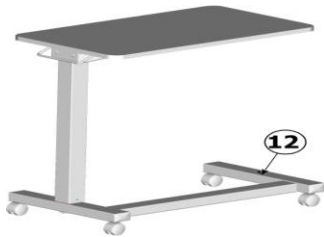

**CAUTION SPRING  
LOADED ASSEMBLY**  
READ INSTRUCTIONS  
SHEET CAREFULLY  
BEFORE PROCEEDING!

## PARTS LIST

- |   |   |
|---|---|
| 1 | TOP 18 X 32                               |
| 2 | TOP BRACKET (400003-color)                |
| 3 | TRIP HANDLE (482807)                      |
| 4 | BACK MOUNTING PLATE (400063-color)        |
| 5 | 3/8 -16 x 3 1/4" BOLT (2) (BT06)          |
| 6 | COLUMN ASSEMBLY (495091-90n)              |
| 7 | U-BASE ASSEMBLY (452803{See notes below}) |

- |    |                                  |
|----|----------------------------------|
| 8  | TWIN HOODED CASTER (CA06)        |
| 9  | BOLT PLATE (101023)              |
| 10 | 3/8-16 x 2 1/4" BOLTS (2) (BT05) |
| 11 | OPTIONAL VANITY                  |
| 12 | HOLE PLUG (2) (PL05)             |
| 13 | BLACK SHIPPING SCREW             |
| 14 | 4600 CASTER PLUG (460004)        |
| 16 | CUP HOLDERS (CH01pack of 2)      |

## TOOLS REQUIRED

PHILLIPS SCREWDRIVER  
9/16" WRENCH

**NOTES:** Low base (452836/452835) uses a 452822 Bolt Plate and (#10) is replaced with 2 BT22 Bolts 3/8-16 x 1" and CA13 Casters

Fork Base (452837) (#10) is replaced with 2 BT01 3/8-16 x 2"  
Low Fork Base (452816) (#10) is replaced with 2-BT01 3/8-16x2" and uses CA13 Casters

**CUP HOLDERS FOR THE 4928 ARE DISHWASHER SAFE**

## ASSEMBLY INSTRUCTIONS

- 1 Lay (#1) top on table or floor with (#2) Top Bracket as shown
- 2 Insert Top Mounting Bolts (#5) into the Back Plate (#4) and screw the bolts into the top Bracket (#2)
- 3 Insert the Column Assembly (#6) with the shipping screw (#13) facing inwards, as shown. The embossed dimples on the Back Mounting Plate (#4) and the Top Bracket (#2) must fit into the mating holes at the end of the outer column. Tighten the bolts with a 9/16" wrench.
- 4 Carefully install the four Casters (#8) into the Base Assembly (#7)
- 5 Insert two bolts (#10) through the Bolt Plate (#9), and then through the Base Assembly (#7) as shown. Align the bolts with the holes in the bottom of the inner column and tighten with 9/16" wrench.
- 6 Place the complete table upright on the casters.
- 7 Install two Hole Plugs (#12) into the base leg opposite the column.
- 8 Remove the Shipping Screw (#13) and discard.

## OPERATING INSTRUCTIONS

- 1 To raise or lower the top, squeeze the Trip Lever (#2) upward to release the lock and adjust the top to the desired position.
- 2 To lubricate the Column Assembly (#6), raise the top until the Column Assembly (#6) is fully extended. Located the lubrication port (#15) and apply a small amount of spray lubricant or lighter machine oil, repeat as needed.

Rev. A 4/20/15

