

Your Silent Knight resource center. AL-52932

Three easy steps:



Step 1. Place pill(s) into Silent Knight Pill Crusher Pouch. Lift handle and place pouch down between the crusher plates to initiate crushing.



Step 2. Press handle down to crush pill(s). Lift handle and pull pouch upward to half way up the crusher plates.



Step 3. Then press the handle down to activate grinding movement. Press handle down several times to grind pill(s) into fine powder. Repeat if needed. Then remove the pouch and add the medication to an appropriate serving medium for administration.