


Adjustable Bracket for Universal Fit

Slides up and down to adjust
fit for most wheelchairs

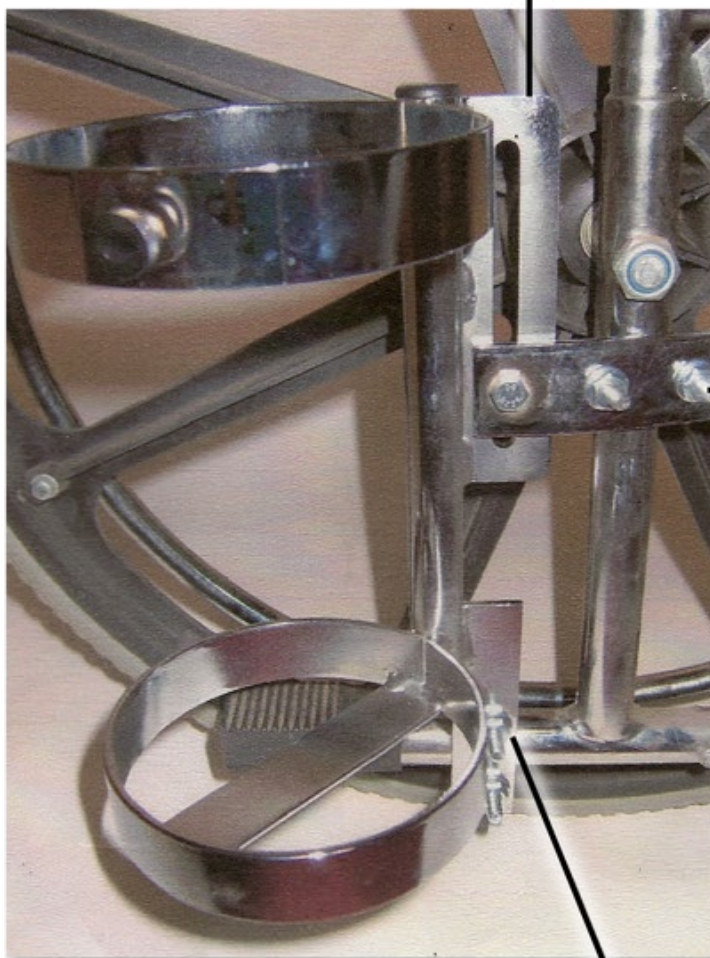

Bolts into
wheelchair

U-bolt to bottom
of wheelchair

INSTALLATION OF PART AL-72294 ONTO WHEELCHAIR

The bottom U-bolt fits over the horizontal bar of wheelchair with bolts facing the inside of chair. Once this is tightened, slide the adjustable bracket up or down to determine the placement you want on the vertical bar of the wheelchair. Tighten the adjustable bracket. (The adjustable bracket allows this unit to fit most wheelchairs.)

Insert the top U-bol around the vertical tube and tighten with the bolts. The screws should be facing toward the inside of the chair.