

## Bath/Emergency Pullcord Station Replacement for Rauland® PCS-113

AL-77017



### Typical Installation

1. Replace existing station with replacement wire for wire. Note the Wire Color Comparison Table for reference.
2. Disconnect header from original Rauland station and connect it to the Replacement.
3. When installing the waterproof pullcord station, ensure the o-rings and gasket are correctly used to ensure a waterproof seal.
4. Always test all nurse call stations for correct operation before placing into service.

### Specifications

<b>ELECTRICAL</b>	
<b>Voltage Rating:</b>	14 VDC @ 100 mA
<b>MECHANICAL</b>	
<b>Terminals:</b>	Wire Leads
<b>Mounting:</b>	Fits on standard single gang back box
<b>SIZE MEASUREMENTS</b>	
<b>Plate:</b>	4.5" x 2.75" (Single Gang)

### Wire Color Comparison Table

Pin	Rauland	Crest	Designation
1	Black	Black	-0V
2	Brown	Brown	CLR/CLA
3	Red	Red	+14V
4	Orange	Orange	Zone Light Common
5	Yellow	Yellow	Bath/Code Common
6	Pink	Grey	Annunciator 3000
7	Blue	Blue	FLASH
8	Violet	Violet	Annunciator III