

TEMPERATURE CHECK AND SANITIZING STATION AL-88527

All-In-One, No contact, No contamination!

FEATURES

- Temperature Check and Sanitizer Station - simple & easy to use
- No contact with hands
- Heavy Duty Stand
- Compatible with hand sanitizer and hand sanitizer gel
- Rust proof durable powder coated aluminum with steel base
- Thermometer checks up to 50 people a minute, Alarm sounds if it detects high temperature, and is available in Celsius or Fahrenheit
- Hand Sanitizer is easy to refill with any soap
- Optional - customize with logo, welcome message, or use instructions. additional cost applies)

TECHNICAL SPECS

- Stand:
Height: 6.58'
Width: 8"
- Base:
Weight- 14 lb
Width- 15.75"
- Thermometer:
Height- 6-3/4"
Width- 4-1/2"
- Hand Sanitizer:
Height- 10"
Width- 4"
- Charger:
6V, 1 Amp
- Batteries:
4 C Batteries



TEMPERATURE CHECK AND SANITIZING STATION AL-88527

INSTALLATION AND USE



STEP 1
Attach rod to base using a countersunk bolt and tighten with the Allen key



STEP 2
Screw the Thermometer onto the pole



STEP 3
Attach knob to stand



STEP 4
Slide the pole into the stand and use knob to adjust the height



STEP 5
Open the Hand Sanitizer with opening clip




STEP 6
Remove Soap Dispenser



STEP 7
Screw the back of Hand Sanitizer to the stand with a screwdriver



STEP 8
Attach nut to the otherside of the stand

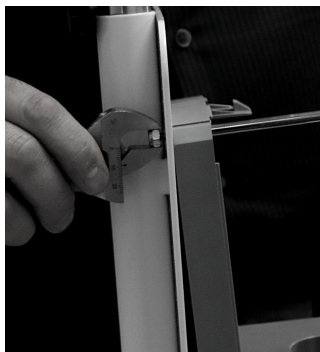

STEP 9
Use adjustable wrench and screwdriver to tighten as needed



STEP 10
Pour any kind of liquid/gel hand sanitizer into soap dispenser



STEP 11
Shut lid of the hand sanitizer to complete installation.

