

## E-Z CALL™ INSTRUCTIONS





AL-46724

### Universal/Quadriplegic Nurse Call Switch

**E-Z CALL** is a sensitive, versatile Nurse Call Switch which can be securely positioned near a patient. It permits almost any patient, even if severely paralyzed, to easily summon aid. E-Z CALL stays where it is placed, ensuring patient accessibility!

E-Z CALL consists of a right-angle frame, with a very sensitive, high-reliability switch mounted in one panel. Over this panel is a floating "Touch-Plate" which operates the switch. E-Z CALL can be positioned in many different ways. The following are only suggestions that you can add to or modify to meet your specific needs.

For the severely paralyzed person, place E-Z CALL near whatever part of the body can most easily be moved (head, arm, hand, knee, foot, etc.) as shown in Fig. 1. The Touch-Plate must face the patient and be close enough to press. Rock the unit away from the patient as shown in Fig. 2 and attach the two clips to the sheet, blanket, pillowcase, etc. Rock it back toward the patient and verify that it is close enough to operate readily, yet distant enough to minimize accidental operations. A variation on this is shown in Fig. 3. For patients with greater range of motion, E-Z CALL can be secured to most bed rails with the "Hook and Loop" strap. See Fig. 4. First, position the unit on the horizontal rail, with the Touch-Plate toward the patient. Then wrap the strap around a vertical member and secure to the *Hook and Loop* fastener.

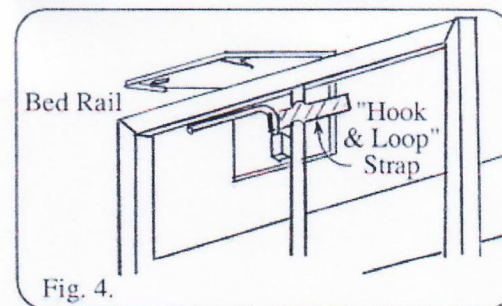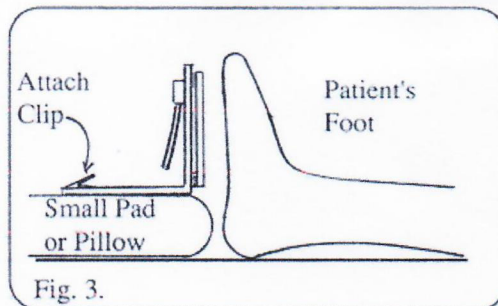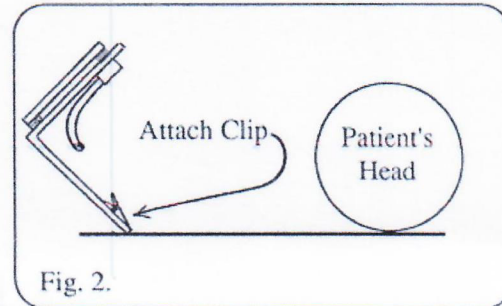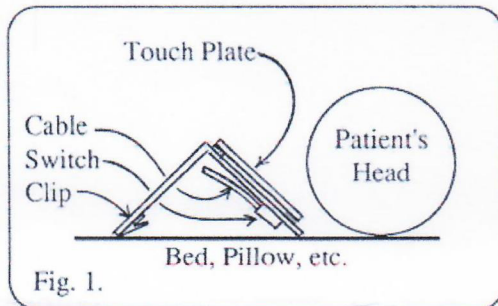

#### WARNINGS:

1. Any equipment by any manufacturer can be damaged or fail to work. Equipment must never be used as a substitute for good nursing and attendant care.
2. Check equipment frequently to verify that it is working properly and that the patient can reach and operate it. This is especially important after repositioning either the patient or E-Z CALL.

#### REPAIR:

If E-Z CALL is damaged or suspected of not working properly, return it to your medical equipment dealer with your receipt and a description of the problem. ALCO will quickly repair the unit if possible and return it to you. If it is still under warranty, there will be no charge except for shipping.

Rev. 1/1/20