




ASSEMBLY INSTRUCTIONS

AL-51335A REPLACEMENT LIDS

NO TOOLS REQUIRED

The replacement lids you are receiving have 2 Lid Pads that have been included for you to install. Depending if you have a Wire hamper or a Tubular hamper will make a difference on where the Lid Pads are installed.

1) Hampers with bag locks: This will include model AL-51333 hampers. The Lid Pads should be installed above the lower groove in the corners of the Lid. See picture # 1. Install both Lid Pads in the same place on opposite sides of the lid. See picture # 2. Peel back the protective backing and place the pads as instructed.

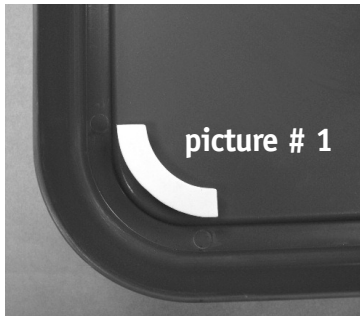

2) Hampers with bag holders: These will include models AL-56644 and AL-56645. The Lid Pads should be installed in the lower groove in the corners of the Lid. See picture # 4. Install both pads in the same place on opposite sides of the Lid. See picture # 3. Peel back the protective backing and place the pads as instructed.



Your Lid is now ready to install onto your hamper.

6851 High Grove Blvd., Burr Ridge, IL 60527
800-323-4282 • Fax 800-950-1167

E-mail: info@alcosales.com www.alcosales.com