

## IDENTIFY COMPONENTS TO START ASSEMBLY



- CLAMP ON FOLDING FOOTREST (FF) WITH WIRE. SCREWSTUD 1.


## STEP 1 DRILL HOLES.



- MEASURE 2" FROM THE TOP OF THE FRONT CORNER FITTINGS BEST TO MARK FRONT AND BACK OF PIPES.
- USE 5/32 DRILL BIT, DRILL HOLES IN THE FRONT AND THE BACK OF THE PIPES.

**NOTE: A PHILLIPS SCREWDRIVER OR A CORDLESS HAND DRILL IS REQUIRED TO TIGHTEN / SECURE ALL SCREWS.**

## STEP 2 – CLAMP ON

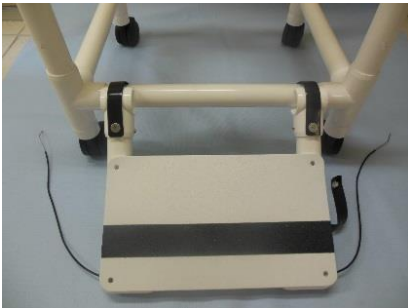


- CLAMP FF ON THE FRONT CROSS BAR.




- SECURE FF WITH THE VINYL STRAPS AND SNAPS PROVIDE.

## STEP 3 WIRE

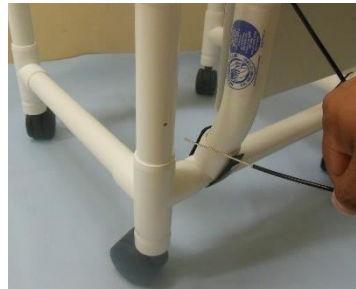


- THREAD THE WIRE THROUGH THE HOLES ON THE VERTICAL PIPES.

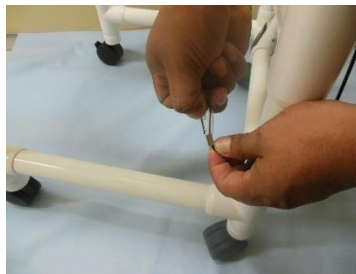

- PLACE THE METAL FERRELS ON THE EXPOSED PART OF THE WIRE FIRST THEN THE CLEAR PLASTIC.

## STEP 4 SLIDING FOOTREST

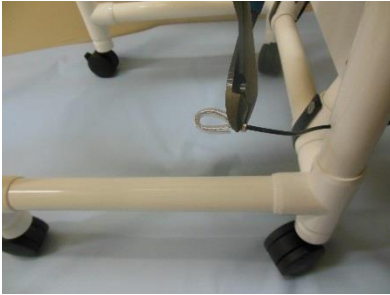

- PUSH THE EXPOSED WIRE THROUGH THE FERREL BEFORE CRIMPING.



- USING A WIRE CUTTER, OR PLYERS TO CRIMP ON THE FERRELS, THIS IS SO THE WIRE DOES NOT COME LOOSE WHEN USING THE FF.

## STEP 5 BACKREST & MESH SLING



- DRILL A HOLE ON THE LEFT TOP CORNER OF THE VERTICAL PIPE (LEFT WHEN SEATING) 1" FROM THE TEE FITTING USE A 1/8 BIT.



- INSERT SCREW STUD.

## STEP 6



- SECURE FF WITH VINYL STRAP.

## STEP 7



- PRODUCT COMPLETELY ASSEMBLED

**NOTE: FF MUST BE CHECKED AND SECURED PRIOR TO USING SHOWER CHAIR.  
WARNING NOT SECURING ALL THE ABOVE CAN CREATE A SERIOUS SAFETY HAZARD.**

## ***PVC PIPING AND CASTERS***

Wipe down piping with non-abrasive cleanser. DO NOT use solvents. Restore luster with Johnson's Pledge or similar spray wax. If casters are used in wet environment, keep lubricated with white lithium paste or similar lubricant; remove hair and mop- thread build up. Provide a weekly maintenance to avoid premature rusting. Rust Proof or Plastic caster need the same care to get the maximum usage.

## ***MESH FABRIC (SLING)***

If laundering is required, use only COLD water in large machine, with regular detergent and one-cup bleach. Do not use softeners or solvents. Spin-dry only; machine heat will shrink mesh fabric. Hang damp fabric on frame and allow to air dry, returning to its original shape.

## ***SOLID FABRIC (PRIVACY SKIRTS)***

Wipe down panels with sponge using solution of mild detergent, one-cup bleach and three gallons water, DO NOT launder in machine. To keep vinyl flexible, occasionally spray with furniture polish (e.g. Pledge) or vinyl formula (Armor All). This will help drying out and cracking.

## ***FOOTREST AND DROP ARMS***

Remove soap build-up from sliding surface; with a soft bristle brush and soapy solution. Lubricate sliding mechanism with furniture spray wax to restore ease in movement.

When lifting drop-arm, use two hands, one at each end of the section. Lifting only at hand grip will put undue pressure against the sliding fitting.

## ***SOFT SEAT***

Wipe with non-abrasive disinfectant wash. Avoid contact with sharp or rough surfaces.

**CAUTION:** DO NOT use chair if casters are rusted and fail to move easily or if cracks occur in fittings.