

Head Positioner

AL Number	Units
AL-62409	Each



Purpose

Provides comfortable lateral head support. Designed to prevent hyperextension of the neck. Attaches to all extended-back wheelchairs, geri-chairs, and recliners.

Easy to Use

1. Place Head Positioner on backrest so that it is just above the patient's shoulders.
2. Bring strap behind backrest and secure buckle. Tighten strap to prevent Head Positioner from slipping.

Easy Care

Head Positioner is covered in durable nylon. It may be wiped clean using a damp cloth and spray-type cleanser.