



Economy Pullcord Station

AL-71104
AL-71105 (With call placed light)

Typical Installation

1. The brown and yellow wires are normally open outputs and are usually connected to the corridor light and the nurse station light.
2. The blue and the green wires are the normally closed contacts and have current available when the switch is in the off (or up) position. Cap all unused wires.
3. The red and orange wires are used for incoming power. The red and orange wires are usually connected together at the time of installation.
4. Cap all unused wires.


Specifications

ELECTRICAL
Max Current and Voltage Rating: 4A @ 30V


MECHANICAL
Mounting: Fits standard single gang electrical back box

SIZE MEASUREMENTS
Plate: 4.5" x 2.75" (1 Gang)

Schematic For AL-71104

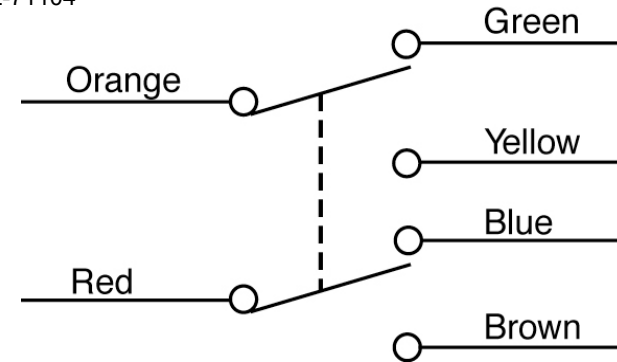

Schematic For AL-71105

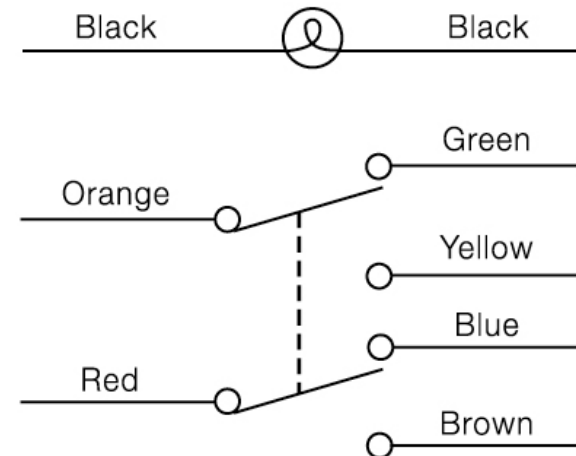

Phone: 800-323-4282

Fax: 800-950-1167



Like us on Facebook at:
www.facebook.com/ALCOSalesServiceCo

Follow us on Twitter at:
www.twitter.com/ALCOSales

info@alcosales.com

www.alcosales.com