


**Adjustable Bracket for Universal Fit**

**Slides up and down to adjust fit for most wheelchairs**

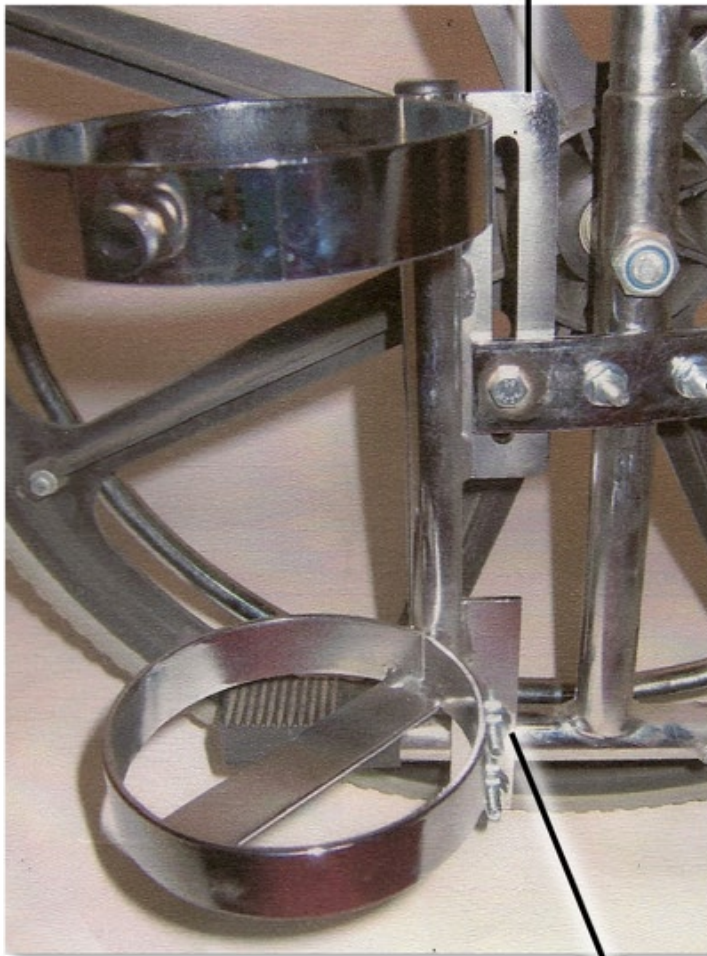

**Bolts into wheelchair**

**U-bolt to bottom of wheelchair**

## INSTALLATION OF PART AL-72294 ONTO WHEELCHAIR

The bottom U-bolt fits over the horizontal bar of wheelchair with bolts facing the inside of chair. Once this is tightened, slide the adjustable bracket up or down to determine the placement you want on the vertical bar of the wheelchair. Tighten the adjustable bracket. (The adjustable bracket allows this unit to fit most wheelchairs.)

Insert the top U-bolt around the vertical tube and tighten with the bolts. The screws should be facing toward the inside of the chair.