


Reservoir Seals Instruction Sheet

Fill reservoir to this line
with bed positioned as:
Head and knee full down
Foot full up and full in
Hi-Lo full up

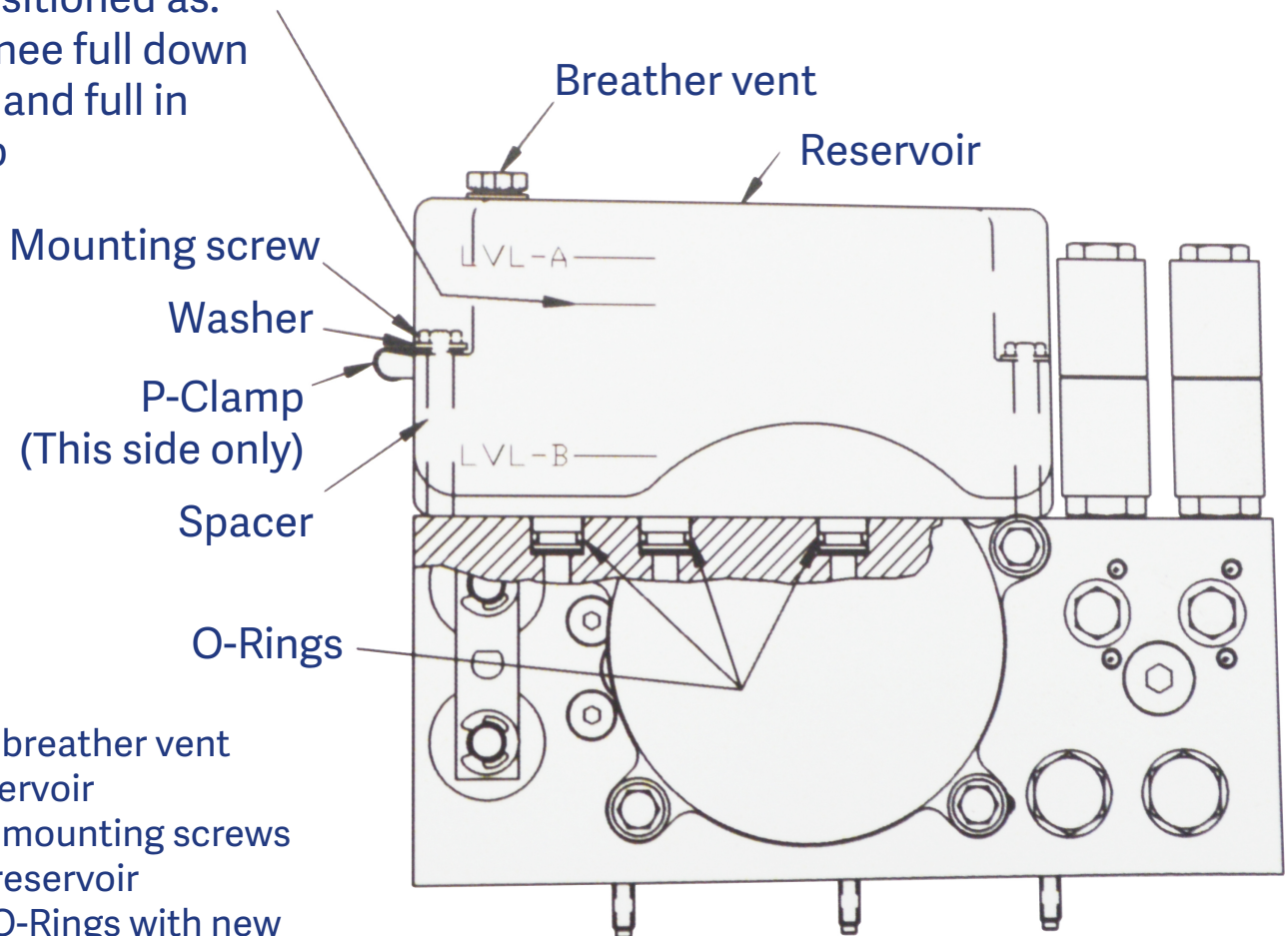

Instructions:

1. Unscrew breather vent
2. Drain reservoir
3. Unscrew mounting screws
4. Remove reservoir
5. Replace O-Rings with new
6. Wipe off top surface of manifold block
7. Install reservoir
8. Install mounting screws (note P-Clamp) (Torque to 5 Ft-Lbs)
9. Refill reservoir with hydraulic oil - Mobil AW32 - to shown level
10. Install breather vent