

ALCO[®]

Sales & Service Co.

AL-83764

Calf Bumpers

TM

Reduce the risk of scrapes and skin tears caused by wheelchair leg rest brackets.

Orient the bumper to the leg rest bracket, pull it open, and slide it on as shown



1



2



3



4

- * Flip bumper over to install on opposite-side brackets.
- * Remove to re-attach leg rests.
- * Clean with soap and water.

Calf Bumpers are our latest product for a wheelchair occupant.

Care facilities frequently remove the leg rest from the chair to allow the occupant to “walk” themselves rather than use their hands on the grip rim.

The exposed brackets that normally hold the leg rest frequently cut into the back of the leg on their calf. {as we get older our skin gets thinner and is prone to tear easily}

An injury has to be reported and documented. All parties must be informed. IE: the physician, family/or power of attorney. All very costly.

Calf Bumpers can prevent the injury and is an excellent “intervention” that has to be included in the written documentation.