

Installation and Use Instructions

AL-84499 Economy Fall Monitor

- Adjustable volume control
- Pad Lost (Patented Technology) light alerts caregivers when a pad is disconnected



AL-84499 Economy Fall/Mobility Monitor

CAUTION: If using this monitor with the Change Pad Indicator™ Timed Sensor Pads, this monitor will not indicate when pad life has expired. The pads will still function normally. Call (800) 650-3637 for more information.

WARNING: Failure to comply with the warning may result in injury or death. This device is not suitable for all individuals. Other devices may be required. This device is not a substitute for visual monitoring by a caregiver. The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. **Test this device before each use. Read the instructions and Legal Disclaimer.**

The AL-84499 Monitor is designed to be installed by the end-user. As such, it is the entire responsibility of the buyer to insure that the system is properly installed and tested. Further, the system is not designed to replace good caregiving practices including, but not limited to:

- Direct patient supervision
- Adequate training for staff personnel for fall prevention and elopement
- Testing of the system before each use

WHERE PERMISSIBLE, IT IS THE SELLER'S INTENT TO LIMIT ANY LIABILITY FOR SPECIAL, CONSEQUENTIAL OR INCIDENTAL ECONOMIC DAMAGES TO REFUND OF THE PURCHASE PRICE OR REPLACEMENT OF THE GOODS. SELLER DISCLAIMS ANY LIABILITY FOR COMPENSATORY OR OTHER DAMAGES ARISING OUT OF THE USE OF THE GOODS.

WARNING: The manufacturer does not claim that this device will stop elopement and/or stop falls. This device is designed to augment caregivers' comprehensive resident mobility management program. This product is to be used by a licensed caregiver only.

Limited Warranty: Seller warrants that the goods sold will be free from defects of workmanship and manufacturing, for a period of one year from the date of sale. Other than this limited warranty, seller makes no express or implied warranties. Except for this limited warranty, NO EXPRESS OR IMPLIED WARRANTIES ARE GIVEN AND NO AFFIRMATION OF SELLER, BY WORDS OR ACTION, WILL CONSTITUTE A WARRANTY. THE GOODS BEING PURCHASED ARE BEING SOLD ON A "WITH ALL FAULTS" BASIS. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE GOODS IS WITH THE BUYER. SHOULD THE GOODS PROVE DEFECTIVE FOLLOWING THEIR PURCHASE, THE BUYER, AND NOT THE MANUFACTURER, DISTRIBUTOR OR RETAILER, ASSUMES THE ENTIRE COST OF ALL NECESSARY INSTALLATION, SERVICING OR REPAIRS.

Bed Sensor Pads

BED PLACEMENT.

- Place the pad under the resident's shoulders to be alerted when the resident sits up in the bed.
- Place the pad under the resident's lower back/buttocks to be alerted when the resident vacates the bed.
- Place the monitor out of reach of the resident. Suitable mounting locations include: the back of headboard, the wall or under the bed. Make sure the resident cannot tamper with the monitor.



TEST THE SYSTEM

- Plug the pad into the monitor and apply pressure to the pad to activate the system. The monitor will beep twice when active.
- The alert will sound when pressure is released from the pad. Move the switch on the side of the monitor to the "OFF" position to silence the alert. Move switch back to the "ON" position to reactivate the monitor.




Chair Sensor Pads

WHEELCHAIR/CHAIR PLACEMENT

- Center the pad on the chair seat.
- Place the monitor out of reach of the resident. Suitable mounting locations include: the back or handle of the wheelchair. Make sure the resident cannot tamper with the monitor.

TEST THE SYSTEM

- Plug the pad into the monitor and apply pressure to the pad to activate the system. The monitor will beep twice when active.
- The alert will sound when pressure is released from the pad. Move the switch on the side of the monitor to the "OFF" position to silence the alert. Move switch back to the "ON" position to reactivate the monitor.

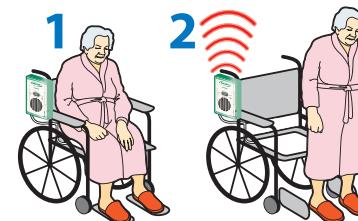

Seat Belt


SEAT BELT PLACEMENT

- Unscrew the one screw on either side of the wheelchair that holds the seat in place.
- Attach the seat belt to the wheelchair by lining up the grommet holes on the seat belt with the screw hole and re-insert screws through both the seat belt and the wheelchair to hold the seat belt in place. Make sure the screws are tightened.
- Adjust the seat belt to the desired length using the black buckle that holds the adjustment strap in place.
- Place the monitor out of the reach of the resident. The suitable mounting location is the back of the wheelchair, so that the resident cannot tamper with the monitor.



TEST THE SYSTEM

- Buckle and unbuckle the seat belt to set off the alert. The monitor is now active. The alert will sound whenever the seat belt is unbuckled.
- Move the switch on the side of the monitor to the "OFF" position to silence the alert. Move switch back to the "ON" position to reactivate the monitor.

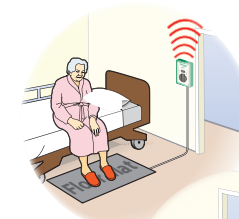

Floor Mats

FLOOR MAT PLACEMENT

- Place the Floor Mat in the desired location such as a critical doorway or next to a bed.
- Place the monitor out of the reach of the resident.

TEST THE SYSTEM

- Step on the Floor Mat to set off the alert. The monitor is now active. The system will sound any time weight is on the Floor Mat.
- Move the switch on the side of the monitor to the "OFF" position to silence the alert. Move switch back to the "ON" position to reactivate the monitor.



Quick Start Instructions




1 Install Batteries

The battery compartment door is secured with the included Phillips-head screw

Tone Adjustment

Alert Volume Adjustment

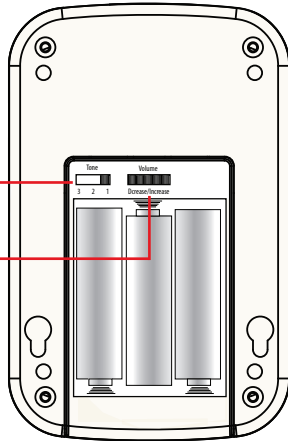


Close battery door and secure it with small Philips-head screw.


Place monitor back into its plastic protective boot.

2 Connect the Pressure Pad

A. Pinch the cord together and route it through the strain relief recess.

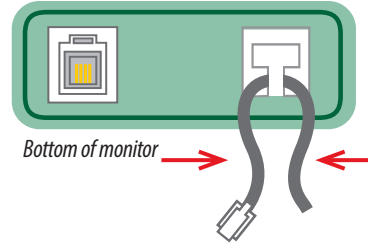


B. Plug the cord into the monitor.

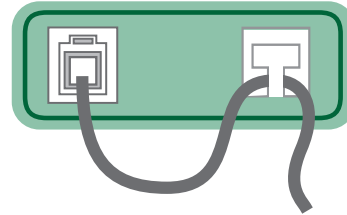

Note: SafeRelease™ cords will not fit into the strain relief recess.

3 Mounting the 3Z* &&+

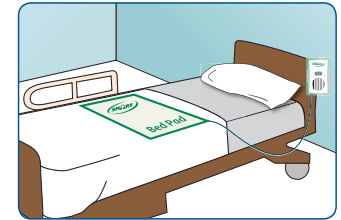


1 Mounting screws in headboard or wall.

2 Hang the monitor on the screws using the mounting holes on back of the monitor.


Monitor may mounted to a bed or a chair with the included locking mounting strap.

4 Test the System - Pressure Pads

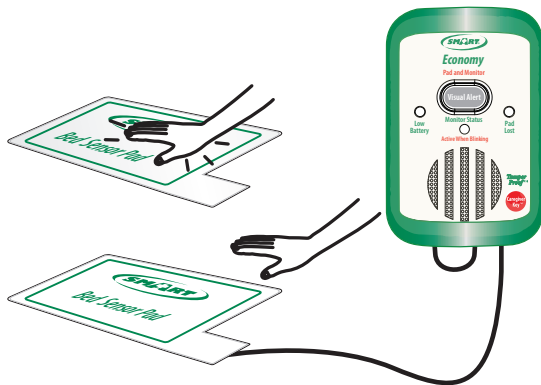

Test the system. (A) Put pressure on the sensor pad to activate the pad and the fall monitor. The fall monitor will beep two times to indicate that the sensor pad and the fall monitor are in use. The "Status" light will blink every 4 seconds while pressure remains on the pad. (B) When pressure is removed, the fall monitor will sound the alert tone. Slide the power switch to "OFF" to silence the monitor. Slide the switch to "ON" to reactivate the system.

5 Test the System - Floor Mats

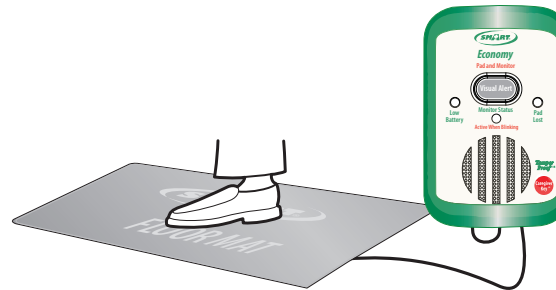


Step on a connected floor mat.

The alert will sound. Slide the power switch to "OFF" to silence the monitor. Slide the switch to "ON" to reactivate the system.

How to Silence and Reset the Monitor

The Visual Alert light blinks when a pad is plugged into he monitor and pressure is on the pad.

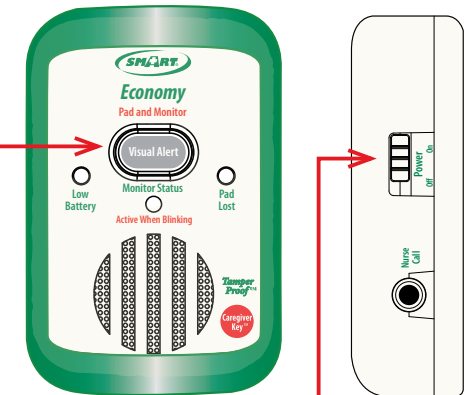

Slide the power switch to "OFF" to silence the monitor. Slide the switch to "ON" to reactivate the system.

