

AL #	Description	Size	U/M
AL-85353-16A	Footrest Extender	16"-18" x 1"	Set
AL-85353-16C	Footrest Extender	16"-18" x 2"	Set
AL-85353-16D	Footrest Extender	16"-18" x 3"	Set
AL-85353-16E	Footrest Extender	16"-18" x 4"	Set
AL-85353-18A	Footrest Extender	18"-20" x 1"	Set
AL-85353-18C	Footrest Extender	18"-20" x 2"	Set
AL-85353-18D	Footrest Extender	18"-20" x 3"	Set
AL-85353-18E	Footrest Extender	18"-20" x 4"	Set
AL-85353-20A	Footrest Extender - Bariatric	20"-22" x 1"	Ea.
AL-85353-20C	Footrest Extender - Bariatric	20"-22" x 2"	Ea.
AL-85353-20D	Footrest Extender - Bariatric	20"-22" x 3"	Set
AL-85353-20E	Footrest Extender - Bariatric	20"-22" x 4"	Set



Note: All models above fit 16"-18", 18"-20" and 20"-22" Wheelchairs and Geri-Chairs

Purpose

The AL-85353 Drop-Stop is designed to solve two important problems: It provides support to control foot drop and it prevents residents' feet from slipping off footrests. The Drop-Stop is designed to fit 16-inch, 18-inch and 20-inch wide wheelchairs and geri-chairs.

Application

1. Before seating resident, lock wheel brakes and rotate footrests to the out-of-the-way position. Assist patient into chair and position him/her as far back on the seat as possible. Reposition footrests.
2. Place Drop-Stop on footrests. Removeable cushion section on footrest, padded backboard against footrest support bars (Fig. 1).
3. Place straps behind footrest support bars, secure buckles and tighten straps (Fig. 2).
4. The Drop-Stop may be positioned to accommodate various degrees of ankle plantar flexion. This may be accomplished by securing the top straps to the upper section of the footrest support bars as shown in the illustration (Fig. 3).
5. Important Safety Note. Remove the Drop-Stop and rotate the footrests to the out-of-the-way position when assisting your resident from the chair. Do not permit your resident to stand with footrests and Drop-Stop in place. Replacement footrest cushions are available in one-, two-, three- and four-inch heights.

Easy Care

Drop-Stop is made from durable vinyl for longlasting use. To clean, just spray and wipe with a damp cloth. Accumulated lint and other particles may be removed from the hook-and-loop material with a small stiff brush.

Fig. 1

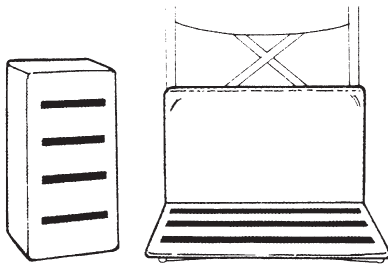


Fig. 2

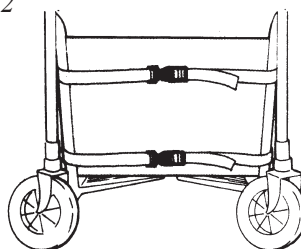


Fig. 3

