

Wheelchair Lap Trays

| AL # | Description | Wheelchair Size | Unit |
|--------------------------------------|---|-----------------|------|
| <input type="checkbox"/> AL-85408-14 | SofTop Tray w/ Vinyl Cover (Pediatric) | 14" | Each |
| <input type="checkbox"/> AL-85408-16 | SofTop Tray w/ Vinyl Cover | 16" - 18" | Each |
| <input type="checkbox"/> AL-85408-20 | SofTop Tray w/ Vinyl Cover | 20" - 22" | Each |
| <input type="checkbox"/> AL-85408-24 | SofTop Tray w/ Nylon Cover (Bariatric) | 24" - 28" | Each |



Purpose

Wheelchair lap trays are designed to serve as therapeutic interventions that assist patients in achieving proper body position, balance and alignment. Although most residents can remove the trays, there are some who can-not. For them, the trays could be considered as restraints.

Instructions

1. Place tray on arms of wheelchair arms. Wrap Velcro™ bands around chair arms and tray. Press the hook material against the loop material to secure.
2. The around-the-back strap is intended to prevent the tray from sliding forward when the resident leans against it. Fasten the strap by bringing it around the back of the wheelchair and passing the end through the buckle. Adjust strap to desired length and secure buckle.

Note: SofTop Trays are reversible. The unpadding side may be used when a firm surface is required.

Easy Care

All trays wipe clean using a damp cloth or sponge.