


Pendant with Motion Control/Nurse Call - AL-85545

The Optional Pendant with Motion Control and Nurse Call was designed to give the patient control over the bed's motion and the ability to contact the nurses from the bed when needed.

The bed must be configured and connected to the hospital system in order for the Nurse Call function to be available.

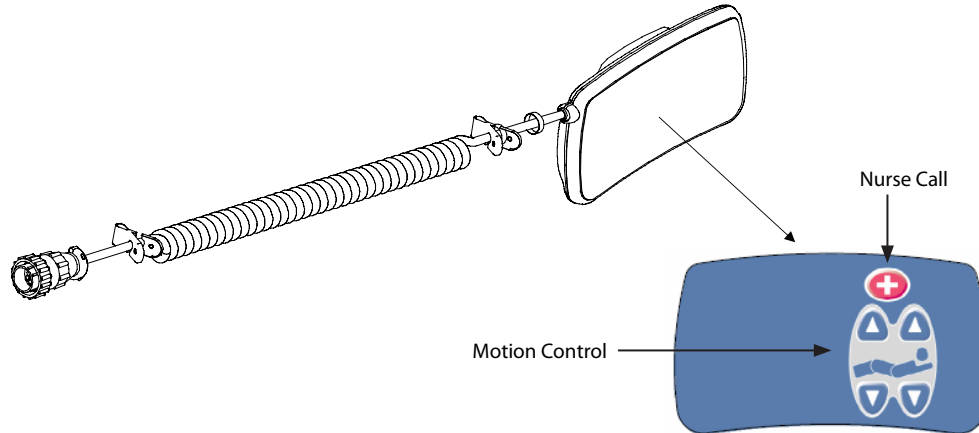

⚠ WARNING


The Weigh system as well as the Bed Exit system must be adjusted if this accessory is added when either system is in function.

INSTALLATION

Required Tools: None

1. Plug the cable connector into connector (A) located on the right of the upper frame.
2. Install the removable pendant on the right foot siderail.

Insert the removable pendant in in this space. Slide it until it clips.

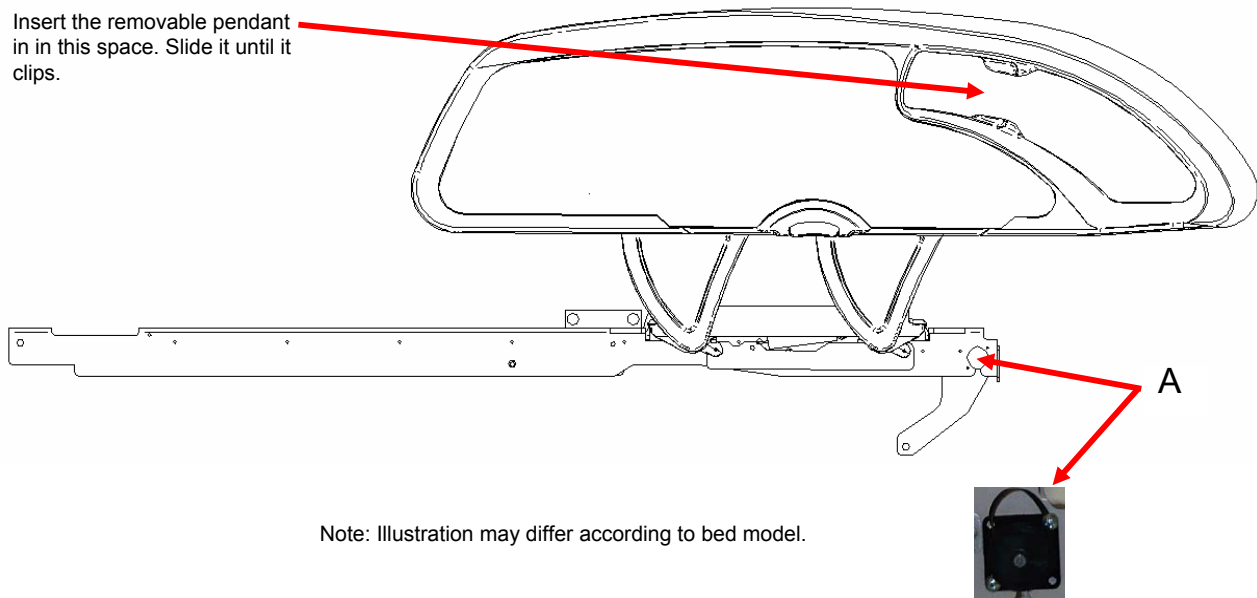

Note: Illustration may differ according to bed model.