



ReOrder	Description	Size	Maximum Weight	U/M
AL-85697	Extra Large Slider Sheet	48"W x 54"L	400lbs	Each

- Low friction bottom that eases positioning and transfer
- Reduces employee musculoskeletal injuries
- Reduces costs associated with patient handling injuries
- Reduces patient falls
- Avoid skin shearing and tears while transferring
- Reduces patients pressure ulcers
- Low Shear stretch, can be placed under patient for extended periods
- Waterproof characteristics, protects bed from body fluids
- Wipe-Clean surface

Purpose

This system is designed for sliding and transferring a patient.

Easy to Use

- Place the slide sheet on the bed facing down
- Lock the bed into position and then position patient on the center of the slider sheet according to your facility's patient handling procedure.

Resident Transfer

Using the handles on the slide sheet, one or two staff members can slide the patient over to a stretcher, gurney, etc.

Caution

- DO NOT use the slide sheets to lift patients. Only use to slide, reposition or transfer patients.
- Patient positioning should be done according to your facility's patient handling procedures.
- DO NOT use with patients who exceed 400lbs in weight.
- Before each use check the slider sheet for signs of wear and replace when necessary.

Easy Care

The slider sheet (not launderable) can be wiped clean using disinfectants.